


BioBlocX™ Reusable Face Mask

Helps reduce exposure to airborne particles

5-Layers

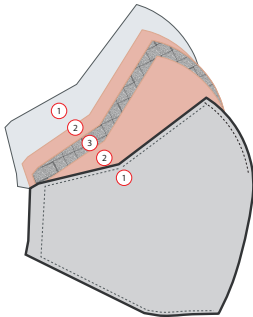

- ① Active Outer Layers: Hypoallergenic CottonX™ creates a durable exterior and a soft, comfortable interior. Fights microbes that attack the mask which can cause product deterioration and odors.
- ② Active Inner Layers: Argaman Accelerated Copper nonwoven fights bacteria with which it is in contact to extend the life of the layer.
- ③ Inner Layer: RESPILON® nanofiber filter membrane, with pores so small that even tiny microorganisms can't get through.

This product provides antimicrobial protection against damaging bacteria, fungus, and microbes that can cause stains, odors, and product deterioration.

The absence of unpleasant odors demonstrates that BioBlocX™ is working to protect fabrics.

External CottonX™ layers powered by EPA-registered material preservative and antimicrobial agent (registration #91367-1).

BioBlocX™ is not NIOSH approved. Not for use against silica, asbestos or other harmful substances. No clinical studies have been conducted comparing the ability of an untreated face mask and these face masks to protect the wearer from infection.



1) Unfold mask by pulling out on chin until mask is fully open.



2) Place chin in the pocket and position the mask on the roof of your nose.



3) Use your other hand to pull the elastic straps over the back of your head.



4) Conform nose piece to nose and adjust position of elastic straps for comfort.



5) Check fit*. If air escapes around your nose, adjust the nose piece. If air leaks at the edges, re-adjust chin and/or tension of the elastic straps.

Cleaning Instructions:

Mask may be washed up to 15 times.

1. Hand wash the mask in warm water with a mild liquid detergent.
2. Rinse thoroughly with drinking quality water and air dry out of direct sunlight.
3. Make sure the mask is completely dry before re-use.

Caution: Do not use bleach, alcohol, cleaning solutions containing bleach or alcohol, or cleaning solutions containing conditioners or moisturizers.

*Facial hair may interfere with face mask fit.

Dispose of mask in accordance with local regulations.

Storage Conditions -4° to 140° F (-20° to +60° C) 15% to 95% relative humidity, non-condensing.

INSTRUCTIONS FOR USE



BioBloc

Smart Prevention

The Natural Protection of Copper



Producer:

Argaman Technologies Ltd.
P.O. Box 10197, Jerusalem 9110102, Israel
phone: +972 2 650 5525
web: www.argamantech.com

Face mask use is not a substitute for hand hygiene.

Limited Warranty :

Argaman Technologies Ltd. ("Argaman") warrants that the product shall be free from defects of workmanship and materials for a period of ninety (90) days from the date of purchase (the "Warranty Period"). If the product fails under normal conditions of use during the Warranty Period and the product is returned to Argaman within the Warranty Period, Argaman will either replace the product or refund the purchase price, at its sole discretion. This warranty is non-transferable and only applies to the original owner of the product. Argaman disclaims all liability for indirect, consequential, special or incidental damages which may be claimed to arise from any sale or use of the product.

ANY IMPLIED WARRANTY, INCLUDING ANY WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, IS SPECIFICALLY DISCLAIMED.

To exercise your rights under this limited warranty, contact Argaman Technologies Ltd. P.O. Box 10197 Jerusalem 9110102 Israel, info@argamantech.com +972-2-650-5525.